EmbryoGen®/ BlastGen™

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en - english

EmbryoGen® is for the culture of human embryos until the 2-8 cell stage. BlastGen™ is for the culture of 4-8 cell stage human embryos to the blastocyst stage. BlastGen™ may also be used for embryo transfer.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Package and pack size
EmbryoGen® (1204): 1 x 3 ml
BlastGen™ (1205): 1 x 3 ml

Contains
Human albumin solution (HAS)
Gentamicin sulphate 10 µg/ml
Sargramostim (GM-CSF) 2 ng/ml
Recombinant human insulin (only BlastGen™)

Quality control testing
Sterility tested (Ph.Eur., USP)
Osmolality tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested < 0.15 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur., USP)
Mouse Embryo Assay (MEA) tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability
The products are aseptically processed and supplied sterile.
Store in original container at 2-8°C, protected from light.
Do not freeze.
Discard excess (unused) media following warming.
The product is to be used within 7 days after opening.
When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings
Do not use the product if:

- Product packaging appears damaged or if the seal is broken.
- Expiry date has been exceeded.
- The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Caution: All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

Caution: This product contains gentamicin and should not be used on patients that have a known allergy to gentamicin or similar antibiotics.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Equilibrate prior to use
Equilibrate media for a minimum of 2 hours in 5-6% CO₂ at 37°C prior to use. The time required to achieve full equilibration depends on the volume of medium and oil overlay.

Instructions for use for EmbryoGen® and BlastGen™

- For IVF cases, fertilise in ORIGIO® Sequential Fert™ or a preferred fertilisation medium. Transfer zygotes from the fertilisation medium to drops or

wells.
Note: Solo utilizar en combinación and return the dish to the incubator.

- For ICSI cases, transfer oocyte from the holding medium to drops or wells of pre-equilibrated EmbryoGen® immediately after the sperm injection and return the dish to the incubator.
- Following initial culture in EmbryoGen®, wash and transfer embryos to fresh microdrops or wells of BlastGen™. Dishes should be returned to the incubator until blastocyst formation on Day 5/Day 6.

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