English

**Quinn's Advantage™
Cleavage Medium**

For laboratory procedures only; other uses must be qualified by the end user.

Product Description	REF Number	Unit Size
Quinn's Advantage™ Cleavage Medium	ART-1026	50 mL
Quinn's Advantage™ Cleavage Medium	ART-1027	100 mL

INTENDED USE
This product was intended for in vitro procedures involving the culture of cleavage- stage human embryos.

PRODUCT DESCRIPTION
This medium is a modification of Human Tubal Fluid (HTF) that was first described by Quinn and colleagues (Fertil Steril. 1984;41:202, 1985;44:493.). Modifications to the original formulation include no phosphate and the addition of a low concentration of glucose, citrate, EDTA, selected nonessential amino acids and taurine, alanyl- glutamine as a stable source of glutamine, increased levels of magnesium, and the presence of lactate in the form of calcium lactate. This product

contains 10 mg/L of gentamicin, an aminogly- coside antibiotic.

PRECAUTIONS AND WARNINGS
Do not use medium that shows evidence of particulate matter, cloudiness, or is not rose colored.
Single Use: To avoid problems with contami- nation, handle using aseptic techniques and discard any excess product that remains in the bottle or vial after procedure is completed.
Reproductive media products are intended for single use only. Re-use of reproductive media may result in using a product past its labeled expiration date or increase the risk of microbial contamination in a subsequent procedure if the practitioner fails to utilize adequate aseptic techniques.

Use of expired or microbial contaminated product may result in suboptimal conditions to promote fertilization and/or embryo quality during in-vitro culture. These conditions may result in the failure of the embryo to develop properly or to implant, potentially leading to a failed assisted reproduc- tive procedure.

The products are aspetically processed and supplied sterile.

Note: Embryo is considered a general term. More precisely, SAGE™ considers the period of time initiating when a single diploid cell results from the fusion of male and female genome resulting in zygote formation with subsequent development from repeated mitotic divisions forming a solid mass or morula (typically day 4-5) and after which a fluid-filled cavity develops resulting in blasto- cyst formation (typically day 5-6) ending with embryo implantation that begins the end of the first week and is completed by the end of the second week post conception.

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician (or

properly licensed practitioner).
This product contains the antibiotic gentamicin sulfate. Appropriate precautions should be taken to ensure that the patient is not sensi- tized to this antibiotic.

QUALITY ASSURANCE
One-cell MEA tested and passed with 80% or greater blastocyst. USP Endotoxin tested and passed with <1 EU/mL.
A Certificate of Analysis is available for this product.

DIRECTIONS FOR USE
This is the preferred medium for the culture of cleavage-stage human embryos from the pronuclear stage to 8-cell embryos in vitro. We currently recommend the use of Quinn's Advantage™ Fertilization Medium (REF #1020/1021) for fertilization of oocytes, and Quinn's Advantage™ Blastocyst Medium (REF #1029) for culture from Day 3 to Day 5/6. The media should be supplemented with an exo- genous protein source such as Human Serum Albumin (REF #3001/3003) or Serum Protein Substitute (REF #3010/3011).

Each laboratory should make its own determination of which medium to use for each particular procedure.
Information on specific aspects of IVF, embryo culture, and cryopreservation is available in our Product Catalog.

STORAGE INSTRUCTIONS AND STABILITY
Store unopened containers refrigerated at 2 °C to 8 °C. Warm to incubator (37 °C) tempera- ture and equilibrate with desired atmosphere containing 5% CO₂ prior to use. Do not freeze or expose to temperatures greater than 39 °C.
In order to ensure optimal performance, SAGE Media™ strongly recommends measuring the pH of the medium under laboratory working

conditions and adjusting the level of CO₂ used to attain the desired pH range for optimal embryo development. The desired pH range for Quinn's Advantage Cleavage medium is 7.2 ± 0.1. The product is stable until the expiration date shown on the label.

- A. Remove desired volume of product using aseptic procedures.
- B. Once removed, do not return any volume of product to the original container.
- C. Do not use if the product becomes discolo- red, cloudy, turbid, or shows any evidence of microbial contamination.

RELATED PRODUCTS
ART-1020/1021 Quinn's Advantage™ Fertilization Medium
ART-1029 Quinn's Advantage™ Blastocyst Medium
ART-3001/3003 Human Serum Albumin 100 mg/mL in Normal Saline
ART-3010/3011 Quinn's Advantage™ SPS Serum Protein Substitute
SAGE In Vitro Fertilization™ has a full line of products for the Reproductive Medicine Specialist. Please call or write for specific information or to receive a copy of our current catalog. For technical questions, or to reach our Customer Service Department, call the SAGE™ Support Line.
Quinn's Advantage™ is a trademark of CooperSurgical, Inc.

**Call the SAGE™ SUPPORT LINE:
In the U.S.: (800) 243-2974 International:
(203) 601-9818**

EXPLANATION OF SYMBOLS

 Contains: Gentamicin.

 CooperSurgical, Inc.
95 Corporate Drive
Trumbull, CT 06611 USA

EC REP
ORIGIO a/s
Knardrupvej 2
DK-2760 Måløv
Denmark
www.fertility.coopersurgical.com
Tel: +45 46 79 02 00

Customer Service:
E-mail: sales@coopersurgical.com

Français

**Quinn's Advantage™
Cleavage Medium**
(Milieu de division)

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Quinn's Advantage™ je obchodní známka CooperSurgical, Inc.