Embryo Freezing Pack

Product No.:

1026

Languages:

- bg
- cs
- da
- de
- el
- en
- es
- et
- fr
- hr
- hu
- is
- it
- kk
- lt
- lv
- mk
- nl
- no
- pt
- ro
- ru
- sk
- sl
- sv
- tr
- uk

Customer Service:

E-mail: customer.service@origio.com
Tel.: +45 46 79 02 02 Fax: +45 46 79 03 02

ORIGIO a/s
Knardrupvej 2, DK-2760 Måløv, Denmark
www.origio.com
Tel: +45 46 79 02 00 · Fax: +45 46 79 03 00

a CooperSurgical Company

Ingredientia

1079 Embryo Freezing Pack – Vial 1
Albumini humani solutio; Aqua; CaCl₂; KCl; KH₂PO₄; MgCl₂; NaCl; Na₂HPO₄; SSR

1024 Embryo Freezing Pack – Vial 2
Albumini humani solutio; Aqua; CaCl₂; KCl; KH₂PO₄; MgCl₂; NaCl; Na₂HPO₄; Propylenglycolum; SSR;

1025 Embryo Freezing Pack – Vial 3
Albumini humani solutio; Aqua; CaCl₂; KCl; KH₂PO₄; MgCl₂; NaCl; Na₂HPO₄; Propylenglycolum; Saccharum; SSR

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en - english

For freezing of human zygotes and cleavage stage embryos.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Package

1079 Embryo Freezing Pack (Vial 1)
1024 Embryo Freezing Pack (Vial 2)
1025 Embryo Freezing Pack (Vial 3)

Pack size

10264002: 1 x Vial 1 (2 ml); 1 x Vial 2 (2 ml); 2 x Vial 3 (2 x 2 ml)
10264010: 1 x Vial 1 (10 ml); 1 x Vial 2 (10 ml); 2 x Vial 3 (2 x 10 ml)

Contains

Human albumin solution (HAS)
Recombinant human insulin

Quality control testing

Sterility tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0.1 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur., USP)
Mouse Embryo Assay (MEA) tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability

The products are aseptically processed and supplied sterile. Store in original container at 2-8 °C, protected from light. Do not freeze prior to use. Discard excess (unused) media following warming. The product is provided in vials intended for single use. When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings

Do not use the product if:

- Product packaging appears damaged or if the seal is broken.
- Expiry date has been exceeded.
- The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Caution: The product is for single use and is not to be reused due to risk of contamination.

Caution: All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

Note: Do not place in the CO₂ incubator, as the buffer capacity of the phosphate buffered Embryo Freezing Pack is insufficient for the CO₂ atmosphere. Attention must be paid to the handling of straws at low temperature, as they may thaw very quickly.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use

- Pre-warm the Embryo Freezing Pack for a minimum of 2 hours at room temperature.
- Zygoter or embryos at the 2-4-cell development stage are placed in medium from Vial 1 for 5 minutes at room temperature.
- Transfer zygoter or embryos to medium from Vial 2 and incubate for 10 minutes at room temperature. *Just before the zygotes or embryos are transferred to the medium from Vial 3, the inside of the cryo straws are cleaned with the same solution, but from another dish.*
- Transfer the zygoter or embryos to the medium from Vial 3 for 15 minutes, still at room temperature.
- The medium from Vial 3 is used to load the zygoter or embryos into the straws. Freezing is carried out according to the cooling programme below.
- Cool from room temperature to -7°C at a rate of 2°C per minute.
- Hold for 5 minutes. Manual seeding. Do not seed the straw close to the zygote/embryo and do not drop or shake it. *When the solution is white, seeding has been initiated.*
- Cool from -7°C to -30°C at a rate of 0.3°C per minute.
- Cool from -30°C to -190°C at a rate of 50°C per minute.
- Transfer the straws into liquid nitrogen (LN₂) and store at -196°C.

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