

0638-01

ORIGIO® Sequential Cleav™

Product No.:

8303

8304

Customer Service:

E-mail:

customer.service@origio.com

Tel: +45 46 79 02 02 • Fax: +45 46 79 03 02



ORIGIO a/s

Knardrupvej 2, DK- 2760 Måløv, Denmark

www.origio.com

Tel: +45 46 79 02 00 • Fax: +45 46 79 03 00

Distributor: ORIGIO AUSTRALASIA Pty Ltd

origio

a CooperSurgical Company

Symbols:





Use within 7 days after opening.



Do not use if package is damaged



Discard excess (unused) media following warming.

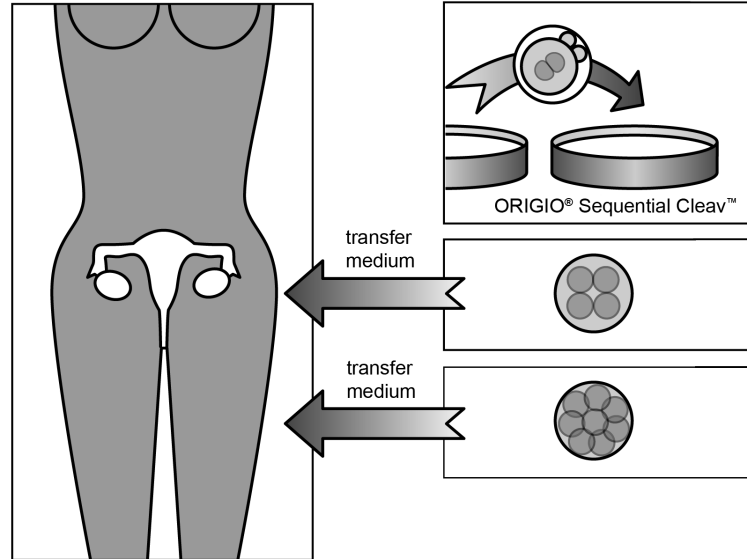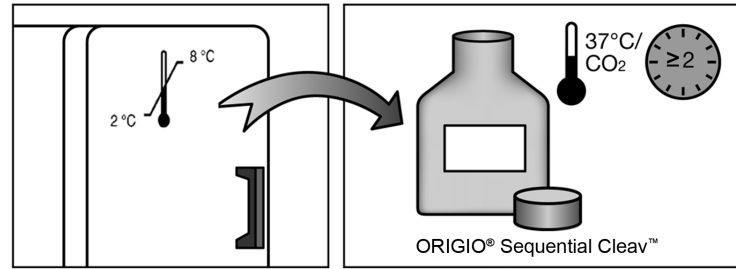

ORIGIO® Sequential Cleav™

For culture until 2-8 cell stage. Can also be used for embryo transfer.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Composition

Human serum albumin (HSA) 5 mg/ml

Gentamicin sulphate 10 µg/ml

Glucose

Sodium pyruvate

Calcium lactate

Physiological salts

Amino acids

L-glutamine (stable form)

Vitamins

EDTA

Sodium hyaluronate

Sodium bicarbonate

Phenol red (only product no.8304).

Quality control testing

Sterility tested (Ph.Eur., USP)

Osmolality tested (Ph.Eur., USP)

pH tested (Ph.Eur., USP)

Endotoxin tested ≤ 0.1 EU/ml (Ph.Eur., USP)

1 cell Mouse Embryo Assay (MEA) $\geq 80\%$

Blastocysts by 96h

Note: The results of each batch are stated on a Certificate of Analysis, which is available on www.origio.com.

Storage instructions and stability

The products are aseptically processed and supplied sterile.

Store in original container at 2-8°C, protected from light.

Do not freeze.

Discard excess (unused) media following warming.

The product is to be used within 7 days after opening.

When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings

Do not use the product if:

1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.

Caution: All blood products should be treated as potentially infectious. Source material from which this product was derived was found negative when tested for antibodies to HIV, HCV, and non-reactive for HBsAg, HCV RNA and HIV-1 RNA. No known test methods can offer assurance that products derived from human blood will not transmit infectious agents.

The potential risk of reproductive or developmental toxicity due to the use of ART media has not been determined and is still unknown.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use

1. Equilibrate for a minimum of 2 hours in 5-6% CO₂ at 37°C prior to use. The time required to achieve full equilibration depends on volume of medium and oil overlay.
2. Transfer zygotes from the fertilization medium to fresh drops or wells of pre-equilibrated ORIGIO® Sequential Cleav™ and return the dish to the incubator.

Embryo transfer

1. The embryos are prepared and transferred to the uterus in pre-equilibrated fresh ORIGIO® Sequential Cleav™ or preferred transfer medium.
2. Flush the transfer catheter with the chosen transfer medium prior to use.

Each laboratory should make its own determination of which medium to use for each particular procedure.